

# PD2 ASSEMBLY INSTRUCTIONS



Fig. 1

## Contents:


1. VitaScan LT Bladder Scanner Probe - Refer to the Quick Start that accompanied the Scanner Probe for instructions on how to scan.
2. VitaScan PD Console
3. VitaScan LT Rolling Medical Cart with Probe Holder and Basket

Once fully assembled and adjusted your cart should look like Fig. 1

## 1


### ASSEMBLE THE LOWER HALF OF THE CART

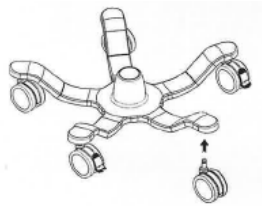


Install the casters onto the base.



#### Install the Post onto the Base

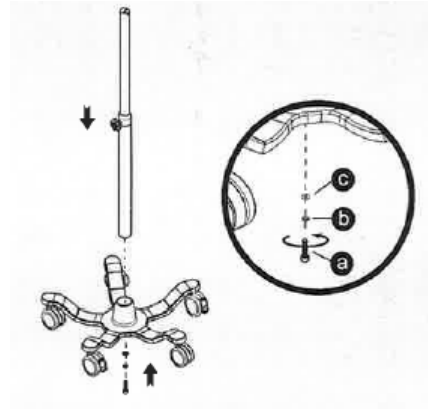Insert the Roll Stand Post (bottom) into Base. Lay assembly on its side for access to bottom base and connect the two using the supplied hardware as shown. Return stand to upright position.



a	b	c
M10 x 50	M10	M10
 x1	 x1	 x1

## 2

### ASSEMBLE THE UPPER HALF OF THE CART

Extend the top post and tighten the adjustment knob to hold it in place.

**Attach mounting bracket to basket.** Using the supplied wrench, securely attach the Basket's Mounting Bracket to the back of the Basket. Loosen the Basket's Mounting Bracket leaving enough gap to slide the Bracket onto the Upper Post.



#### Install the Basket onto the *upper* post.

Slide the Basket onto the upper post from the top down. Position Basket about midway down the post and tighten the screws. Adjust for ideal positioning once fully assembled.

3


### MOUNT THE PD CONSOLE TO THE CART

Attach the PD's Mounting Bracket to the back of the PD Console using the four Black Capped Screws.



Insert the stem of the mount into the Upper Post. Align the holes and secure using the three included screws.

Use the supplied wrench to remove the two screws and washers from the PD Console and use them to attach the Probe Holder.

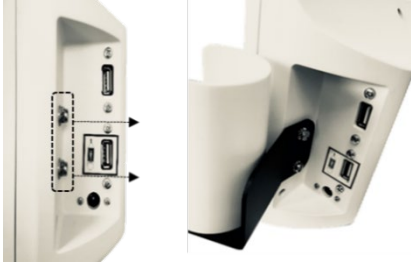

*Adjust the Basket positioning so that it is aligned with the monitor, as shown in Fig. 1*

4

### CONNECT POWER CORD & SCANNER PROBE TO CONSOLE

Mount the Cord Cleat to the back of the Basket's mounting bracket.

Connect the power cord to the PD Console and use the Cord Cleat to organize the power cord.



Connect the Scanner Probe to the PD console and then place the Scanner Probe into the Probe Holder.

Note, for added security place the Scanner's cord into the cord lock mechanism and twist the mechanism into the locked position.



**Allow the PD Console to fully charge before first use. Charge PD Console when not in use.**



It is recommended to operate this equipment on battery power. To avoid the risk of electrical shock, use only properly grounded equipment. Grounding reliability can only be achieved when equipment is connected to a receptacle marked "Hospital Grade" or equivalent. Run Tablet on battery whenever the integrity of the external protective earth conductor arrangement is in doubt.

**Call 800-757-8450, option 2 for assistance.**