


PD2 ASSEMBLY INSTRUCTIONS


Contents:

1. VitaScan LT Bladder Scanner Probe (refer to the Quick Start that accompanied the Scanner Probe for instructions on how to scan)
2. VitaScan PD Console
3. Rolling Cart with Cord Cleat and Basket


1 ASSEMBLE THE LOWER HALF OF THE CART

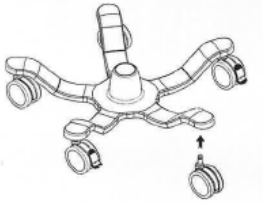

1.a Install the casters onto the base.

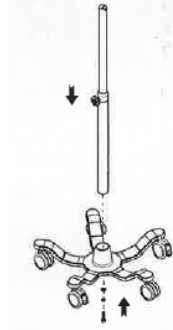

1.b Insert the bottom post into Base.

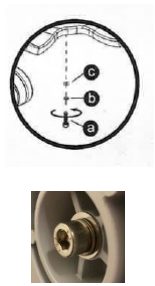

a	M10 x 50
b	M10
c	M10

1.c Lay assembly on its side to access the base bottom. Grip the post to prevent rotation. Secure the post to the base using the Hex Wrench. Return stand to upright position.



It is important that you fully secure the bolt using the Hex Wrench so that it is flush. Do not hand tighten.

2 ASSEMBLE THE UPPER HALF OF THE CART



2.a Extend the top post and tighten the Adjustment Knob to hold it in place.



2.b Attach the Basket's Mounting Bracket to the Basket. Loosen the Bracket leaving enough gap to slide the Bracket onto the upper post.



2.c Slide the Basket onto the upper post opposite the Adjustment Knob. Position the Basket approx. midway down the post and tighten the bracket evenly.




2.d Attach the Cord Cleat to the back of the Basket's Mounting Bracket using the top two holes. Insert the Velcro strap with the fuzzy side on the outside.



Install with the "lip" at the bottom (as shown).

3 ASSEMBLE THE PD AND MOUNT TO THE CART

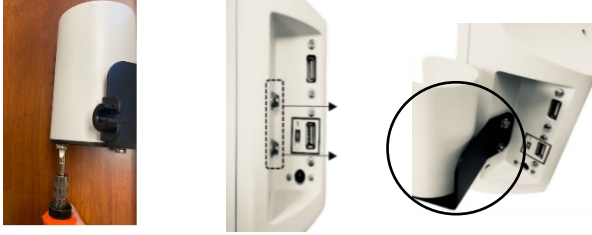

3.a Using the supplied wrench, attach the Probe Holder Sleeve to the Bracket and then the Bracket assembly to the PD Console.




3.b Connect the PD to the Cart Mounting Bracket using the 4 Black Capped Screws.




3.c Insert the stem of the Cart Mounting Bracket into the upper post and align with the post holes. Secure using the 3 supplied screws. Swivel the PD Monitor side-to-side until aligned with the Basket.




3.d Connect the Scanner Probe to the PD's USB Port and then place the Scanner Probe into the Probe Holder.

 For added security place the Probe's cord into the cord lock mechanism and twist the mechanism into the locked position.

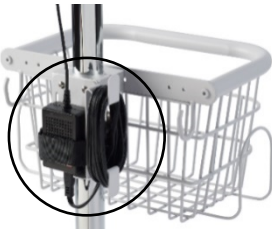


3.e Connect the power cord to the PD Console and use the Cord Cleat to organize the power cord.

 Install so that the LED light is visible and the Power Brick is resting on the "lip". Tightly secure the Velcro.



3.f The cart assembly is complete. Your scanner should closely match the image above.

 If the cart is "wobbly" ensure the post is properly secured to the base (see step 1.c).

4 CHARGE SCANNER



Allow the PD to fully charge before first use. Always charge the PD when not being used.

It is recommended to operate this equipment on battery power. To avoid the risk of electrical shock, use only properly grounded equipment. Grounding reliability can only be achieved when equipment is connected to a receptacle marked "Hospital Grade" or equivalent. Run PD on battery whenever the integrity of the external protective earth conductor arrangement is in doubt.