

## VITASCAN PD2 BLADDER SCANNER INCLUDES:

- (1) VitaScan LT Bladder Scanner Probe - Refer to the Quick Start that accompanied the scanner probe for instructions on how to scan.
- (1) VitaScan PD Console
- (1) Rolling Medical Cart with Basket/Handle


## ASSEMBLE THE CART BASE AND POST

- ① Install the Casters onto the Base.




Connect the Lower Post to the Base. Lay assembly on its side for access to the bottom of the Base.

- ② Connect the two using the supplied hardware. Return stand to upright position. Extend the Upper Post to the preferred height and tighten using the adjustment knob to hold it in place.

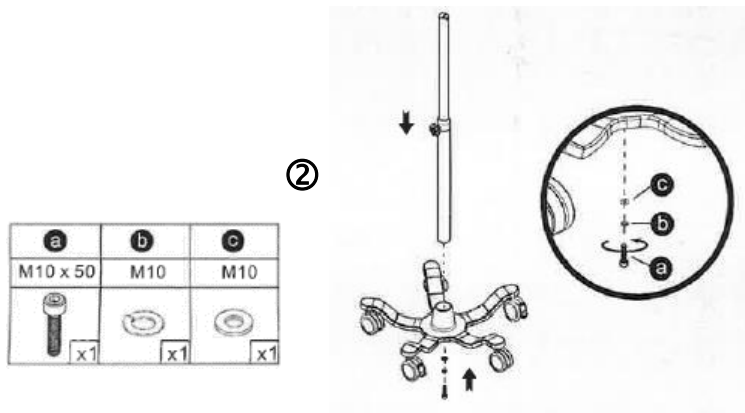


## ASSEMBLE AND ATTACH THE BASKET TO CART

- ③ Using the supplied wrench, securely attach the Basket's Mounting Bracket to the back of the Basket.

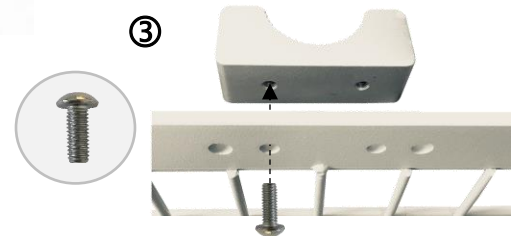


- ④ Using the supplied wrench, loosen the Basket's Mounting Bracket leaving enough gap to slide the Bracket onto the Upper Post.

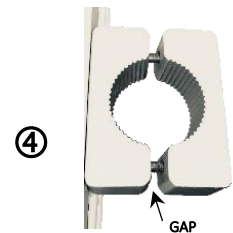

- ⑤ Slide the Basket onto the Upper Post from the top down. Position about midway down the Post and tighten the Mounting Bracket. You can adjust for ideal positioning once fully assembled.



- ⑥ Attach the Power Brick Mount to the back of the Basket's Mounting Bracket.

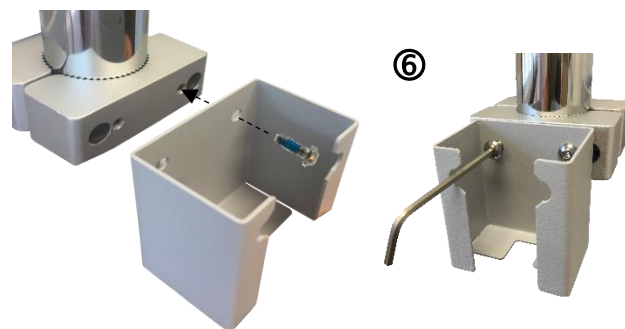


## ASSEMBLE CONSOLE AND ATTACH TO CART


- ⑦ Attach the PD's Mount to the back of the PD Console using the four Black Capped Screws.
- ⑧ Insert the stem of the mount into the Upper Post. Align the holes and secure using the three included screws.



**⚠** *Once assembled and adjusted your scanner should look like Fig. 1*  
*Fully charge the PD Console prior to using.*

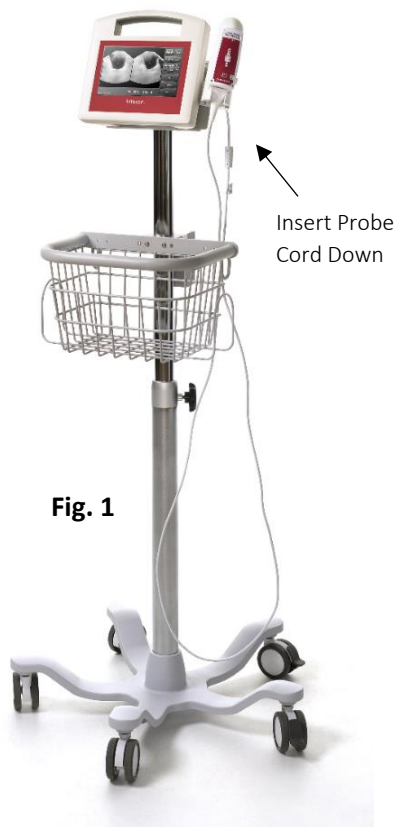

**Fig. 1**

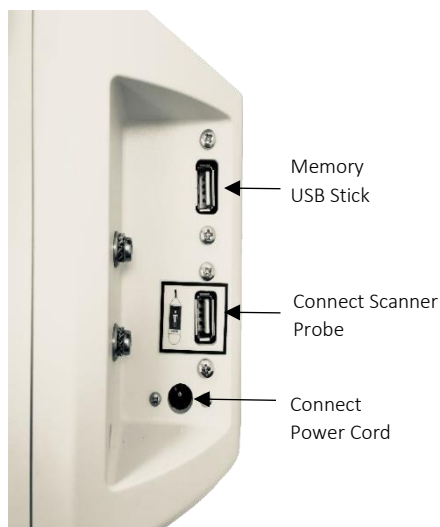

**CALL 800.757.8450 (OPTION 2) FOR ASSISTANCE.**