

VitaScan LT Bladder Scanner Cleaning Instructions – C6 Medical Grade Systems

General Cleaning

In general terms, you can **clean** the entire system with a suitable cleaning solution mixture of 50% Isopropyl alcohol and 50% distilled or otherwise purified water.

Disinfecting

We recommend PDI Sani® Cloth AF3 Germicidal Disposable Wipes, or other comparable disposable wipe designed for use on non-porous plastic surfaces.

Wipe down each component and allow to *air-dry*.

You may also use any glutaraldehyde-based hospital disinfection solution or Clorox Healthcare bleach products. Use appropriate hand protection according to the labeling on the disinfectant to avoid skin reactions.

1. Dampen a soft cloth in any glutaraldehyde-based hospital solution.
2. Wipe the instrument thoroughly.
3. To remove all traces of disinfection solution, wipe the VitaScan LT with a clean soft cloth dampened in sterile water.
4. Carefully dry the VitaScan LT with a clean soft cloth before use.

Warning!

- Do not subject any part of the VitaScan LT to steam sterilization or ethylene oxide sterilization.
- Do not immerse the instrument in any cleaning or disinfecting solution.
- Do not use CidexPlus® to disinfect the instrument. CidexPlus® will damage the plastic enclosure.
- Do not thermally disinfect.
- Do not use abrasive cleaning aids, sponge pads, or microfiber cloths for cleaning or disinfecting.